



## Instructions for Use

Your Makana Mask uses our unique specialty filters.

- Remove the additional filters from the package.
- Put the extra filters into the anti-static bag (included).
- Close the zip lock seal for maximum filter life and performance.
- Each filter is designed to last 3 continuous days of use.

## To Use the Mask

- Place the mask over your nose and mouth.
- The top of the mask should rest gently across the nose.
- The bottom of the mask fits on the chin just below the mouth.
- Pull straps over your head.
- Adjust the straps to ensure a snug fit.

*Your Makana Mask is ready to use.*

## To Clean your Makana Mask

Soap and Water Method (RECOMMENDED)

- Remove the filter following the instructions below.
- Wash the facemask and filter cap in hot soapy water.
- Dry the mask and filter cap with a clean cloth or towel.

To Sanitize

- Remove the filter following the instructions below.
- Apply Isopropyl alcohol (rubbing alcohol) to a paper towel or clean cloth.
- Wipe the exterior and interior of the mask.
- Allow to air dry.

## Filter Removal and Replacement

- Gently squeeze the filter cap and pull to release from the mask.
- Remove the used filter and discard.
- Insert a new filter into the filter cap.
- Snap the filter cap onto the mask.

*Your Makana Mask is ready to use.*

# Makana



## OHANA FILTER

Breathability ●●●●●

Our Ohana filter is our most comfortable option. We recommend the Ohana for everyday use for essentials errands, school, sports and camp.



## MAHALO FILTER

Breathability ●●●○○

Our Mahalo filter is great when you require a little more protection. It works especially well for crowded environments, commuter transit and commerce.



## ALOHA FILTER

Breathability ●●○○○

The Aloha is our densest electrostatic filter, which lasts a bit longer. It is great for construction, factory and industrial use.



## MĀLAMA FILTER

Breathability ●●○○○

The Mālama filter is an activated carbon cloth with silver for added antiviral protection. It is great for front line healthcare and airline travel.